


# Quick Set Infusion Set Abbreviated Instructions


**Note:** This is an abbreviated user guide for general reference. Staff should familiarise themselves with the full user instructions enclosed in each box as this may contain additional user and safety information.

## Priming the Quick-set

Using a Luer-lok compatible syringe, connect the medication to the tubing.

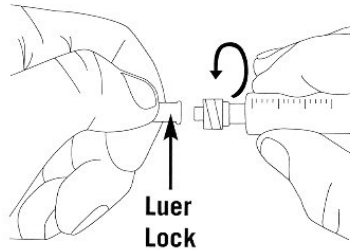


Advance the syringe plunger gently until drops form at the needle end.

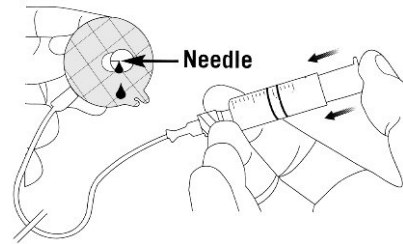

## Inserting with the Quick-serter


1. Select a suitable site for infusion, paying attention to the depth of insertion dependent on the catheter length chosen. Exercise caution where there may be a risk of contact with bone on insertion. Prepare the site according to standard protocol.

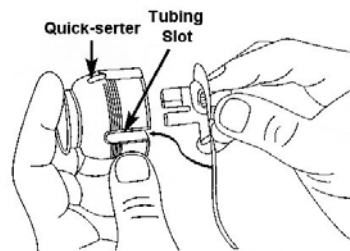


2. Align tubing with tubing slot and place set into Quick-serter.




3. Remove paper backing from adhesive tape.

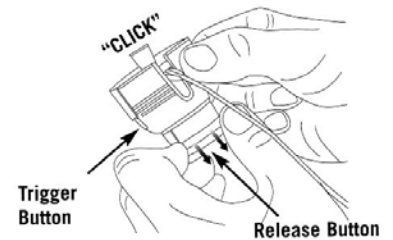


4. Pull back the release button until it clicks into place in activated position.

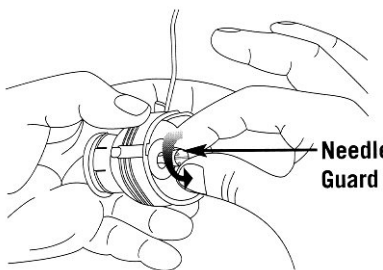


5. Twist to remove needle guard and retain for later use.

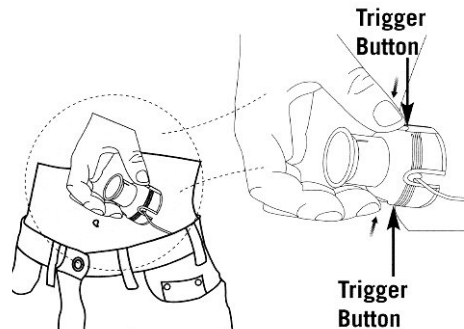


6. Place the Set over the selected infusion site, and push the two white trigger buttons at the same time to release and insert the set.

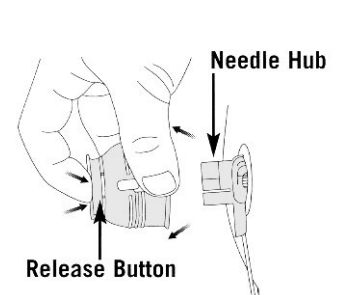

7. Press the release button to detach the Quick-serter and lift away from needle hub. Smooth down adhesive tape.

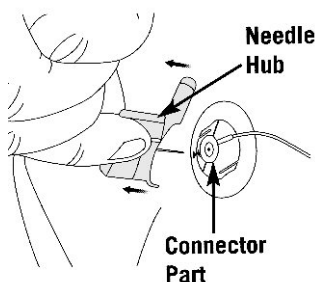


8. Gently pull the blue needle hub away from connector part.

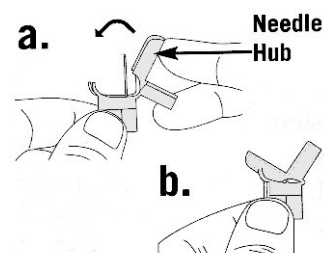


9. Fold needle hub until it snaps into place, and dispose into sharps container.

## Inserting the Quick-set Manually


1. Select a suitable site for infusion, paying attention to the depth of insertion dependent on the catheter length chosen. Exercise caution where there may be a risk of contact with bone on insertion. Prepare the site according to standard protocol.

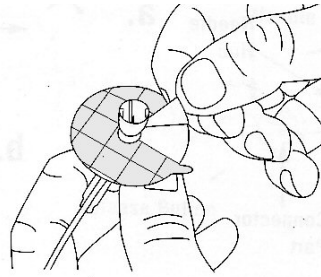


2. Remove paper backing from adhesive pad.

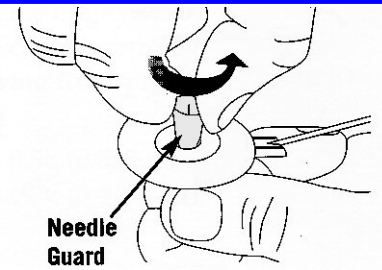


3. Twist to remove needle guard and retain for later use.

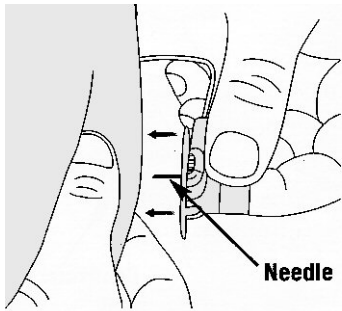

4. Pinch skin and insert the Quick-set at a 90 degree angle. Smooth down the adhesive.

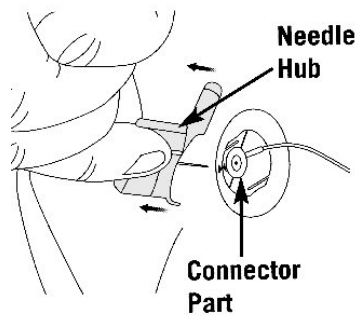


5. Gently pull the blue needle hub away from connector part.

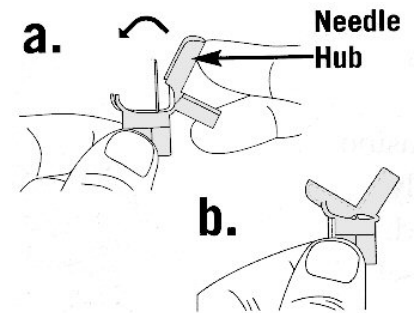


6. Fold needle hub until it snaps into place, and dispose into sharps container.


## Disconnecting the Quick-set

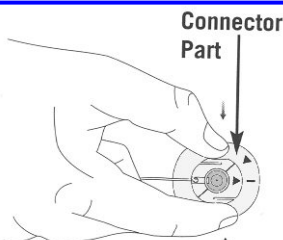


1. Gently squeeze grips on connector part.

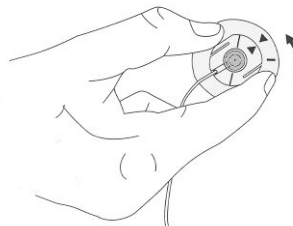

2. Rotate connector part to align triangles.

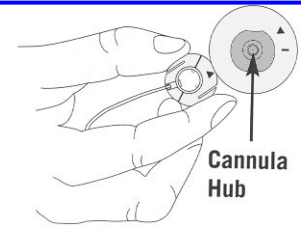


3. Lift off connector part from cannula hub.

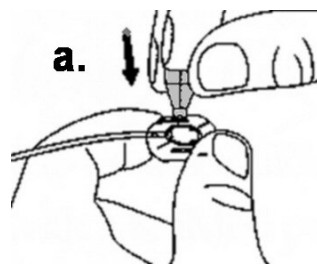

4. For storage purposes:

- a. Insert needle guard into connector part.

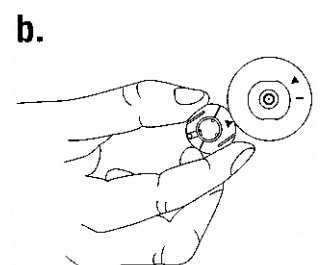


- b. Place protective cap on cannula hub.


## Reconnecting the Quick-set

### Before reconnecting:

- Remove needle guard from connector part
- Remove protective cap from cannula hub

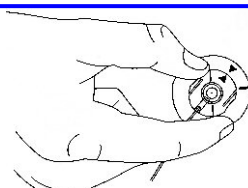

1. Gently hold ridges of connector part, and place onto cannula hub with triangles aligned.

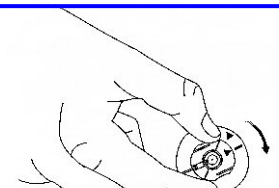

2. Rotate connector part until you hear or feel a 'click' and the triangle aligns with the line.